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KANGHUA SAFETY EQUIPMENT MANUFACTURING CO., LTD
Dianhouzhou Village, Zhuge Town, Lanxi City
Zhejiang, China

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**User instruction
Face shield, MZ-5****Description**

These eye protectors are face shield with plastic lens.

Field of Use (Protection capabilities and performance characteristics)

These products are intended for protection against liquid splashes, which are likely to damage the eye or impair vision.

Service life

The visor should be replaced after one years in service.

Storage and Maintenance

If possible, store the product in a container when not using it.

Cleaning is recommended after each use.

The product should be cleaned using a cloth moistened with warm soapy water and dried at room temperature. A suitable lens cleaning cloth may also be used.

Do not use petrol, chlorinated degreasing fluids, organic solvents or abrasive cleaning agents to clean any part of the equipment.

The product can be disinfected with alcohol or disinfectant fluid regularly.

Accessories and spare parts

Accessories: protective covers to the lens, user manual

There is no spare parts.

Standards and Approval

The Face shield system are classed as Personal Protective Equipment (PPE) by the European PPE new regulation (EU) 2016/425 and have been shown to comply with this regulation through the Harmonized European Standard EN 166:2001.

Marking

HKH 1 3 CE

Manufacturer s code: HKH

Optical class: 1

Protection against liquid splashes: 3

Use Limitation and warning

1. This visor is not suitable to protect against any mechanical impact.
2. Never modify or alter this product.
3. Do not use this product against hazards other than those specified in this document/product.
4. Scratches or damages oculars should be replaced.
5. Materials, which may come into contact with the wearer`s skin could cause allergic reactions to susceptible individuals.
6. The compatibility of the marking shall be ensured or the protection afforded by the goggles may be impaired.

Manufacturer information

KANGHUA SAFETY EQUIPMENT MANUFACTURING CO. LTD
Dianhouzhou Village, Zhuge Town, Lanxi City,
Zhejiang, China

Declaration of conformity

Hereby, KANGHUA declares that the face shields are in compliance with Regulation (EU) 2016/425.
The full text of the EU declaration of conformity is available at the following internet address:

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ECS GmbH (priglašeni organ št. 1883)