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Insérer le clip d'identification.

GB
Universal dispenser
for hand disinfectants and
liquid soap

wall mounted

Remove wall bracket.
Press the catch and slide off
the bracket downwards.

Take notice of distances
above and below the dispenser.
Use fixing set enclosed.

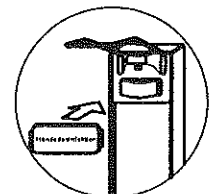
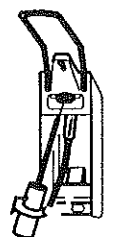
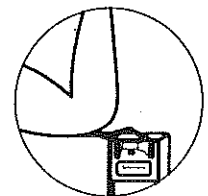
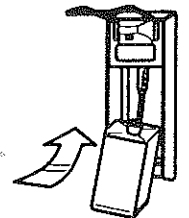
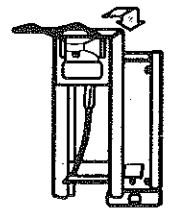
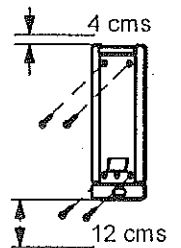
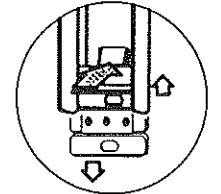
Put the dispenser on the bracket.
The catch will click in and secures
the dispenser.

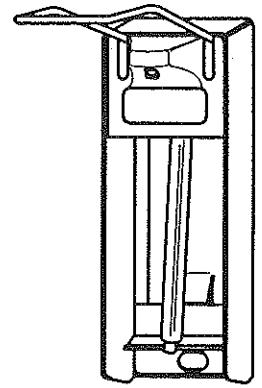
Insert filled bottle and operate the
lever a few times until dosage is
delivered.

The hygienic elbow operation.

The optimized dosage is adjusted
ex works.
Further adjustments can be done to
reduce dosage.
Use screw driver (cross no.1)
and turn 1/8 counter clock wise.

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Insérer le clip d'identification.

GB

Universal dispenser
 for hand disinfectants and
 liquid soap

wall mounted

Remove wall bracket.
 Press the catch and slide off
 the bracket downwards.

Take notice of distances
 above and below the dispenser.
 Use fixing set enclosed.

Put the dispenser on the bracket.
 The catch will click in and secures
 the dispenser.

Insert filled bottle and operate the
 lever a few times until dosage is
 delivered.

The hygienic elbow operation.

The optimized dosage is adjusted
 ex works.
 Further adjustments can be done to
 reduce dosage.
 Use screw driver (cross no.1)
 and turn 1/8 counter clock wise.

Change clip if necessary.